






















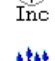




Name of Program: ANAAbV2
Instrument:

BlotWareVersion: V1.00
BlotServer Version: V1.10 / 1, 1, 0, 20

Type:	Num.:	Title:	Parameters
 Disp	1	Dispense	Channel = 1; Volume = 2500 µl; FirstStrip = SetAtStart; NumStrips = TillEnd
 Inc	2	Incubate	Incubate: Time = 1 min. Shake: Speed = 4
 Pause	3	Pause	
 Inc	4	Incubate	Incubate: Time = 60 min. Shake: Speed = 4
 Wash	5	Wash	Channel = 2; Volume = 2500 µl; FirstStrip = Begin; NumStrips = TillEnd
 Inc	6	Incubate	Incubate: Time = 5 min. Shake: Speed = 4
 Wash	7	Wash	Channel = 2; Volume = 2500 µl; FirstStrip = Begin; NumStrips = TillEnd
 Inc	8	Incubate	Incubate: Time = 5 min. Shake: Speed = 4
 Wash	9	Wash	Channel = 2; Volume = 2500 µl; FirstStrip = Begin; NumStrips = TillEnd
 Inc	10	Incubate	Incubate: Time = 5 min. Shake: Speed = 4
 Asp	11	Aspirate	
 Disp	12	Dispense	Channel = 4; Volume = 2500 µl; FirstStrip = Begin; NumStrips = TillEnd
 Inc	13	Incubate	Incubate: Time = 30 min. Shake: Speed = 4
 Wash	14	Wash	Channel = 2; Volume = 2500 µl; FirstStrip = Begin; NumStrips = TillEnd
 Inc	15	Incubate	Incubate: Time = 5 min. Shake: Speed = 4
 Wash	16	Wash	Channel = 2; Volume = 2500 µl; FirstStrip = Begin; NumStrips = TillEnd
 Inc	17	Incubate	Incubate: Time = 5 min. Shake: Speed = 4
 Wash	18	Wash	Channel = 3; Volume = 2500 µl; FirstStrip = Begin; NumStrips = TillEnd
 Inc	19	Incubate	Incubate: Time = 5 min. Shake: Speed = 4
 Asp	20	Aspirate	

Type:	Num.:	Title:	Parameters
 Disp	21	Dispense	Channel = 6; Volume = 2500 µl; FirstStrip = Begin; NumStrips = TillEnd
 Inc	22	Incubate	Incubate: Time = 30 min. Shake: Speed = 4
 Asp	23	Aspirate	
 Disp	24	Dispense	Channel = 5; Volume = 2500 µl; FirstStrip = Begin; NumStrips = TillEnd
 Inc	25	Incubate	Incubate: Time = 20 min. Shake: Speed = 4
 Asp	26	Aspirate	